


## MYSOZO CLINICIAN

# Changing Measurement Units

1

## Select Settings


1. Sign into MySOZO as a clinician
2. From the header menu select "Settings"
3. Sidebar menu will appear

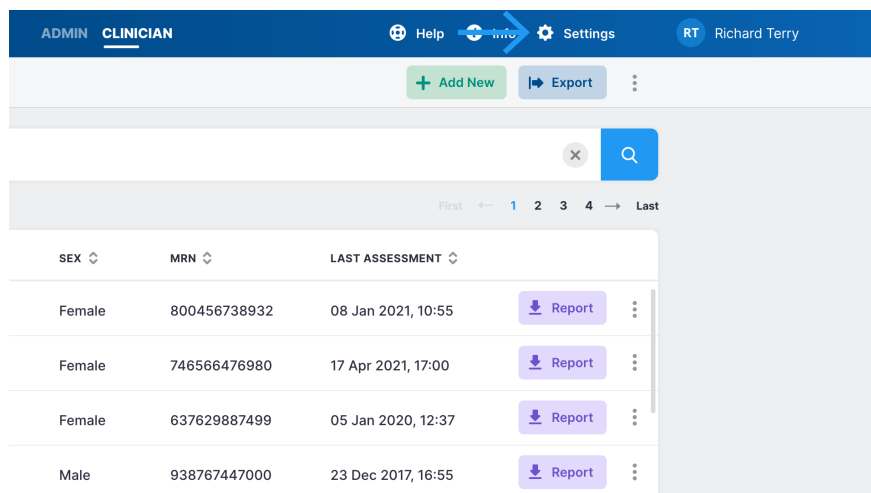

2

## Select Measurement Units


1. From the sidebar select "Measurement Units"

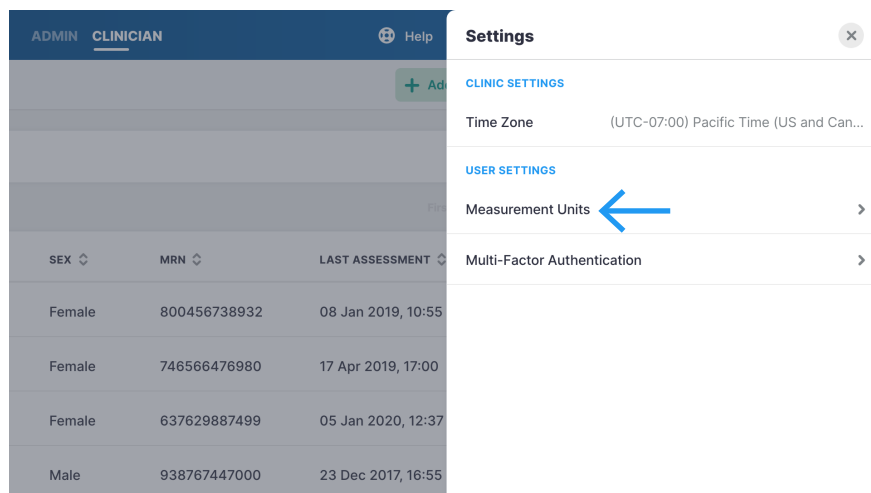

3

## Select Units

1. Select preferred units of measurement
2. Select "Save"

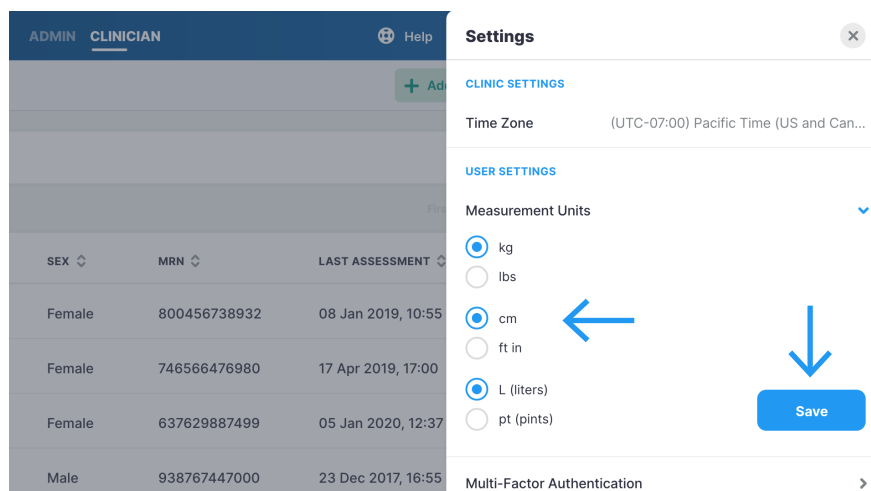


## SOZO APP

# Changing Measurement Units

1

## Select Settings


1. Sign into SOZOapp as a clinician
2. From the bottom navigation menu select "Settings"
3. Sidebar menu will appear

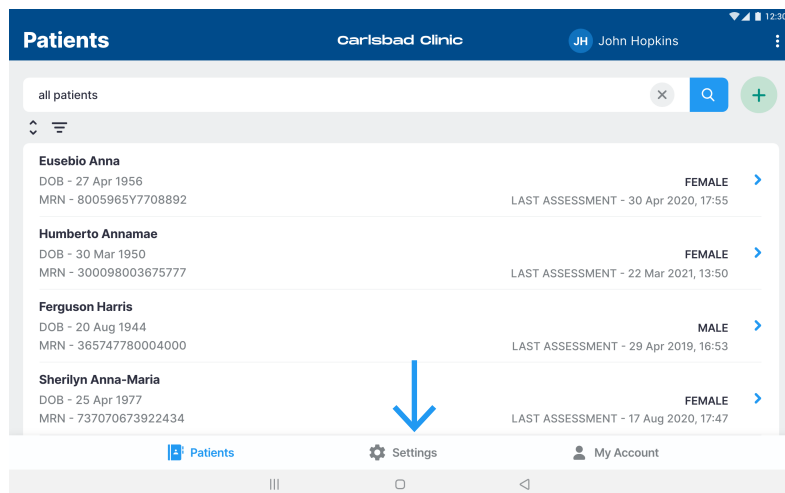

2

## Select Measurement Units

1. From the sidebar select "Measurement Units"

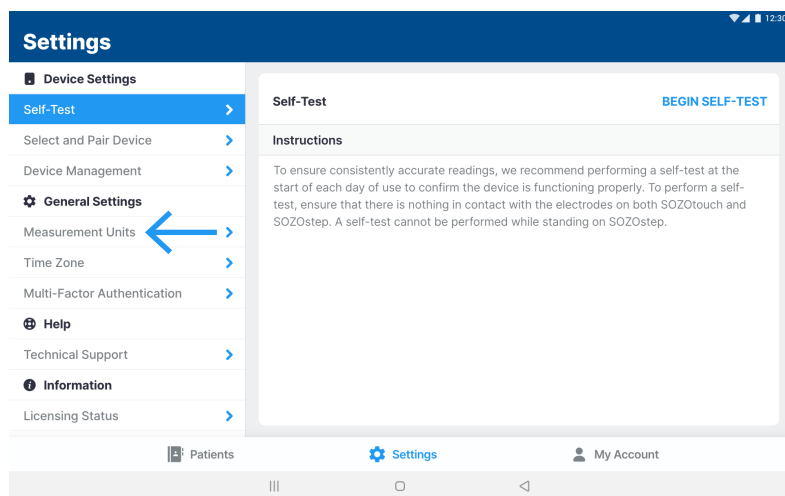

3

## Select Units

1. Select preferred units of measurement
2. Select "Save"

