

INTRODUCTION:

The SOZO Software Suite, v3.1.0 was released in April of 2020 with the purpose of introducing new features. With these added features, a bug was introduced affecting the already existing baseline functionality of ECF and Hy-Dex when using SOZOapp – SFT-013 and SFT-027.

ISSUE DESCRIPTION:

The issue impacts *the baseline of the ECF baseline* within the Fluid Assessment module *and the Hy-Dex baseline within the Tissue assessment module (within the US) or in the Fluid Assessment Model (outside the US)* under the following scenario:

- Within the SOZOapp, the user navigates to a patient’s history screen and selects the Fluid Assessment output.
- The user then selects a measurement to be set as a baseline for the ECF output.
- At this moment, all graphical outputs and measurement results are shown and calculated as expected.
- When the user navigates away from the measurement and then back to the measurement, the measurement used for the baseline that was set earlier was recorded as the Hy-Dex baseline rather than the intended ECF baseline. The Hy-Dex change from baseline calculation and graphical display are now based on the new incorrect measurement that was selected. The ECF baseline has not been updated.
- If the baseline is set from within MySOZO.com, it performs as intended.

AFFECTED PRODUCTS:

- | | | |
|---|---|--------------------------------|
| <input checked="" type="checkbox"/> SOZO Analyzer | <input checked="" type="checkbox"/> Software Version: 3.1.0 | <input type="checkbox"/> U400 |
| <input type="checkbox"/> SFB7 | <input type="checkbox"/> Electrodes (gel backed) | <input type="checkbox"/> Other |

INSTRUCTIONS:

To prevent the bug from occurring; it is recommended to set the ECF baseline from within MySOZO.com and not through the SOZOapp for software version 3.1.0 only.

CONTACT INFORMATION:

Australia: ImpediMed Limited Unit 1, 50 Parker Court Pinkenba Qld 4008 Australia Tel: +61 (0)7 3860 3700 Free call: 1800 638 477 Email: enquiries@impedimed.com	USA: ImpediMed Inc. 5900 Pasteur Court, Suite 125 Carlsbad, CA 92008 United States Tel: 760 585 2100 Toll-free: 877 247 0111 Email: info@impedimed.com
---	--