

## INTRODUCTION:

The latest version of MySOZO v4.1 added an Analytics module that allows clinicians to view and export L-Dex related data at a clinic and patient level. The Analytics dashboard has several tabs - LPP Protocol Compliance, Program Management, Demographics, and **Patient List**. For more information, please refer to the Definitions tab within the Analytics dashboard.


## ISSUE DESCRIPTION:

When sorting on Last or First Name in the Patient List of the Analytics dashboard, it first sorts on uppercase A-Z then on lowercase a-z.

LAST NAME	FIRST NAME	DOB	SEX	MRN	SURGERY DATE	LAST ASSESSMENT
Alessa	Demo	Nov 28, 1963	Female	34512ty	Apr 30, 2021	May 19, 2021
Alice	Wonderland	Oct 26, 1986	Female	4444444	Sep 23, 2019	Oct 2, 2020
Apple	jimmy	Aug 8, 1996	Female	45123	null	Apr 7, 2021
Bat	Women	Oct 30, 1985	Female	123123	null	Mar 27, 2020
Bird	Tweety	Jul 16, 1962	Female	1435	null	May 14, 2021
Brittain	Tammy	Aug 21, 1962	Female	4321	null	Mar 26, 2021
C	Heather	Mar 11, 1990	Female	2016	null	Jun 24, 2021
Demo	Shelly	Oct 17, 1978	Female	9889	null	Jul 19, 2021
Edwards	A	Dec 16, 2004	Female	de1234567	null	Mar 13, 2021
Edwards	R	Mar 21, 1968	Male	de12345678	null	Mar 13, 2021
Edwards	S	Dec 29, 1974	Female	de123456	null	Mar 13, 2021
Lavine	Adam	Mar 18, 1979	Male	5374	null	May 20, 2021
Maleficent	.	Oct 31, 1981	Female	24242424	null	Sep 19, 2019
Ogden	Debra	Jan 1, 1973	Female	12df	null	May 8, 2021
Test	Christy	Mar 22, 1978	Female	00000	null	Mar 23, 2021
Winner	March	Feb 16, 1976	Female	121212	null	Jun 17, 2021
Wright	E	Feb 7, 1977	Female	1223345	null	May 7, 2021
Wright	R	Apr 18, 1957	Female	de1234	null	Mar 13, 2021
adams	mollie	Jun 14, 1980	Female	541234	Apr 1, 2021	Apr 9, 2021
dabis	diammy	Feb 2, 1977	Female	5555589	null	Jun 24, 2021
mae	sally	May 21, 1963	Female	345sdf	null	May 18, 2021

## RESOLUTION:

As a workaround, to sort patients last or first names without considering capitalization (similar to how it sorted in Excel), click on 3 dots on top right corner, click on **Export to Excel** and sort the desired column of your downloaded Excel document.

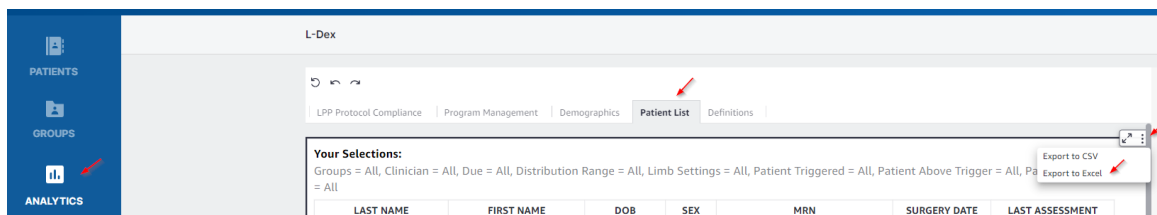

## AFFECTED PRODUCTS:

- SOZO Analyzer
- Software Version: 4.1
- U400
- SFB7
- Electrodes (gel backed)
- Other

## CONTACT INFORMATION:

<p><b>Australia:</b> ImpediMed Limited Unit 1, 50 Parker Court Pinkenba Qld 4008 Australia Tel: +61 (0)7 3860 3700, Option 2 Free call: 1800 638 477 Email: <a href="mailto:ts@impedimed.com">ts@impedimed.com</a></p>	<p><b>USA:</b> ImpediMed Inc. 5900 Pasteur Court, Suite 125 Carlsbad, CA 92008 United States Tel: 760 585 2125 Toll-free: 877 247 0111, Option 4 Email: <a href="mailto:tsu@impedimed.com">tsu@impedimed.com</a></p>
<p><b>Europe:</b> ImpediMed Hellas Tel: +30 231-111-6753 Email: <a href="mailto:tse@impedimed.com">tse@impedimed.com</a></p>	