

## SOZO APP

# Quick Start Guide

Ensure you have fully read and understood the complete MySOZO instructions for use available at [www.impedimed.com](http://www.impedimed.com).


Tip: Contact your MySOZO Administrator for account set-up details.

1

## Sign In

1. Sign into SOZOapp as a clinician


Username is your email address created by your MySOZO Administrator.

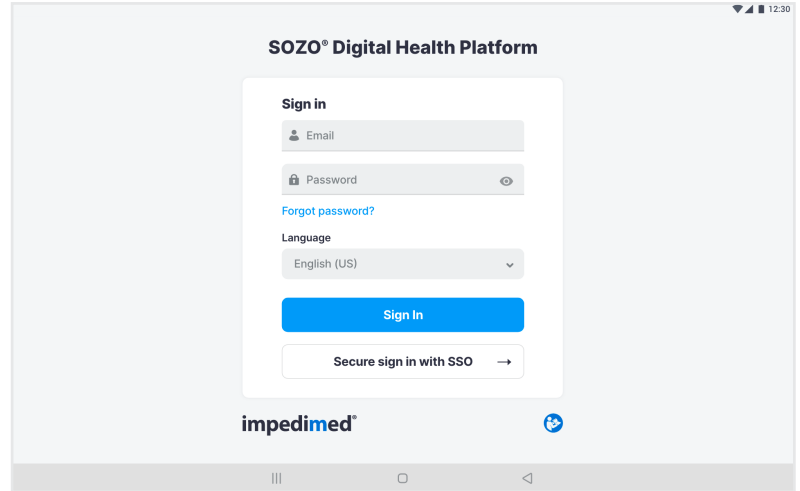

2

## Patient List

1. Provides access to a list of patients in your MySOZO database
2. Allows searching by Last/First name, DOB, sex, MRN and Last assessment date
3. Provides access to individual patient data, by selecting a patient in the patient list

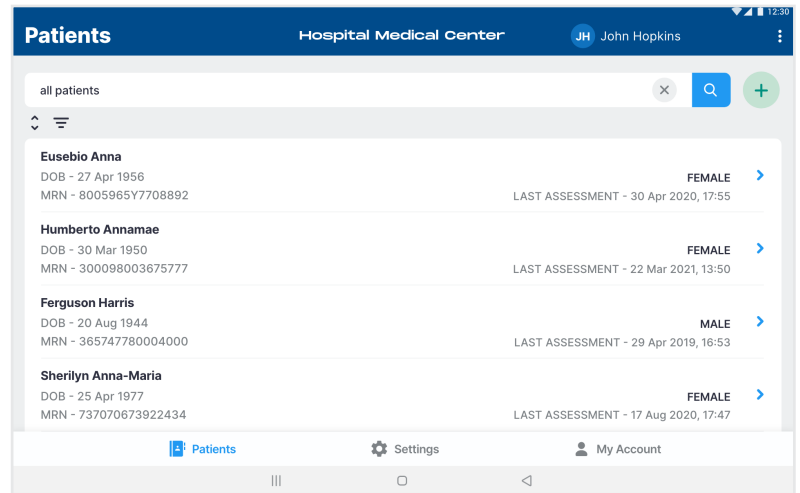


3

## Patient Dashboard

Patient dashboard will give access to:

1. Take a new measurement
2. All measurements taken for the patient along with information about the measurement such as baselines and tags
3. Patient profile
4. Patient notes

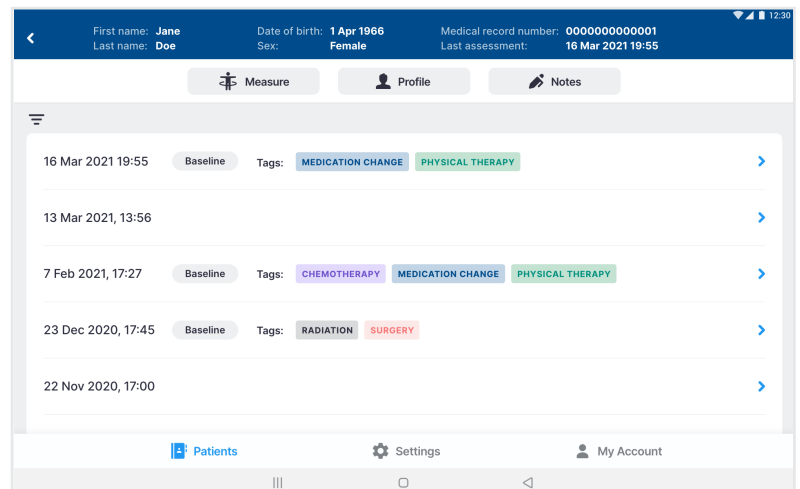

SOZO APP


# Quick Start Guide

4

## Profile

Patient profile will give access to:

1. Demographics
2. Assessment types available
3. Patient groups

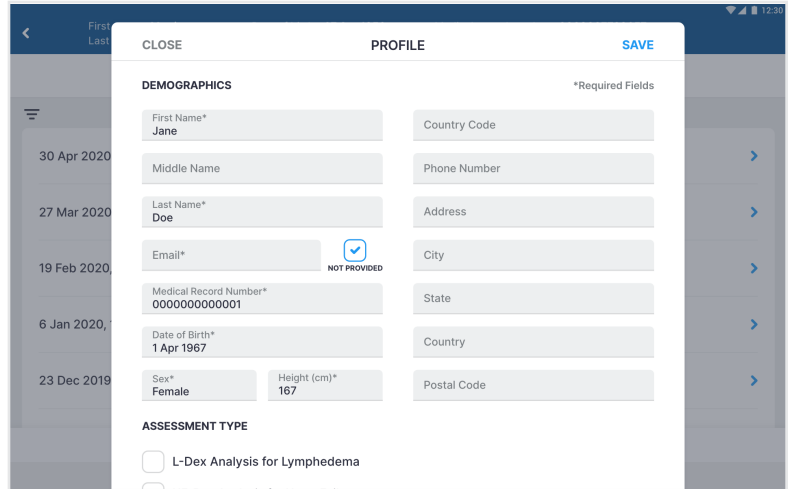


5

## Notes

1. Provides access to a list of notes assigned to the selected patient
2. Ability to create new note
3. Ability to edit or delete an existing note


Please note: A maximum of 10 notes is allowed per patient. To add a new note, first delete an existing note.

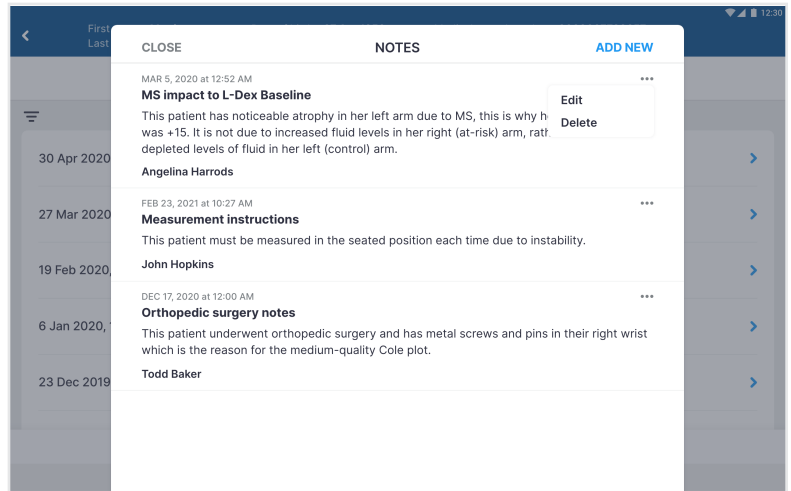

6

## Results

1. Provides access to measurement outputs
2. Provides access to historical data
3. Ability to set baseline to customize trending results
4. Ability to assign measurement tags

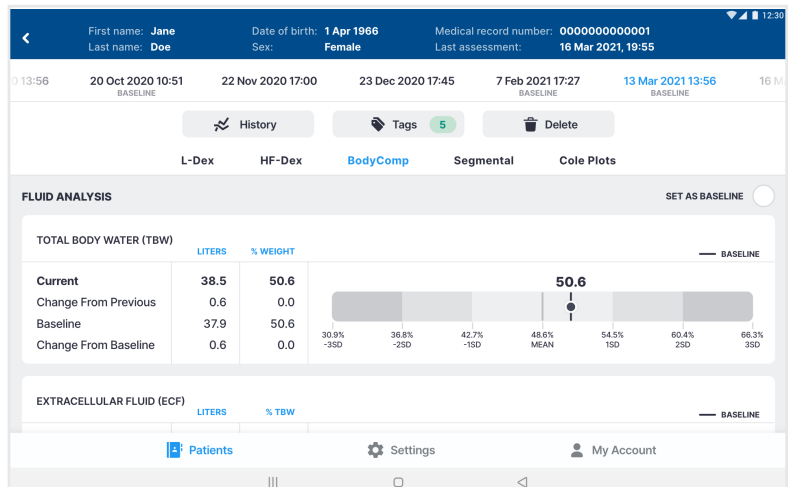


# Quick Start Guide

7

## History

1. Displays graphs of patient measurements overtime.

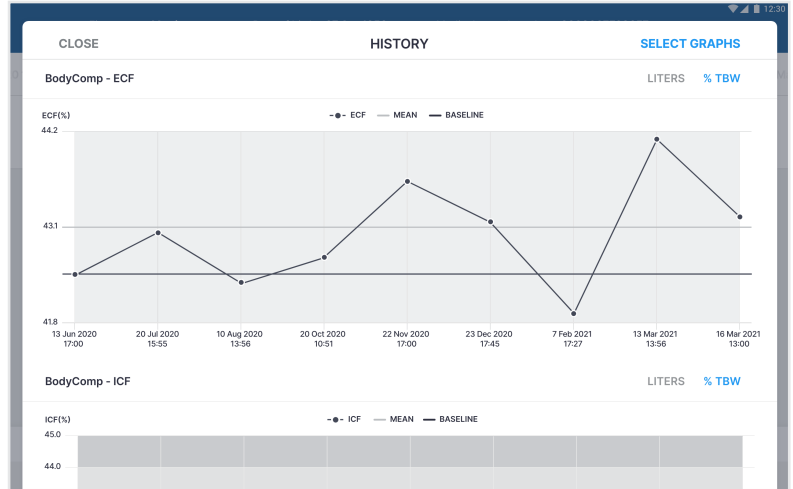

8

## Tags

1. Provides access to a list of tags assigned to the selected measurement
2. Ability to assign new tag
3. Ability to edit or delete an existing tag

Please note: A tag must be first defined by an Administrator before it can be assigned to a measurement.

The screenshot shows a 'TAGS' menu with a list of tags. Each tag includes a date and time, a description of the event, and the patient's name. The tags are: CHEMOTHERAPY (MAR 5, 2020 at 12:52 AM, John Hopkins), MEDICATION CHANGE (FEB 23, 2021 at 10:27 AM, Todd Baker), PHYSICAL THERAPY (DEC 17, 2020 at 12:00 AM, Angelina Harrods), RADIATION (AUG 15, 2020 at 12:17 AM, John Hopkins), and SURGERY (MAY 5, 2020 at 12:52 AM). There are 'Edit' and 'Delete' buttons for each tag.

9

## Restore Patients

1. Provides access to a list of deleted patients
2. Ability to activate a deleted patient

The screenshot shows a 'Restore Patient' screen with a search bar and a list of deleted patients. Each patient entry includes their name, DOB, MRN, gender, and last assessment date. A 'RESTORE' button is visible next to each entry. The patients listed are: Eusebio Anna (DOB - 27 Apr 1956, MRN - 8005965Y7708892, LAST ASSESSMENT - 30 Apr 2020, 17:55), Ferguson Harris (DOB - 20 Aug 1944, MRN - 365747780004000, LAST ASSESSMENT - 29 Apr 2019, 16:53), and Sherilyn Anna-Maria (DOB - 25 Apr 1977, MRN - 737070673922434, LAST ASSESSMENT - 17 Aug 2020, 17:47).