

## SOZO APP

# Quick Start Guide

Ensure you have fully read and understood the complete MySOZO instructions for use available at [www.impedimed.com](http://www.impedimed.com).


Tip: Contact your MySOZO Administrator for account set-up details.

1

## Sign In

1. Sign into SOZOapp as a clinician


Username is your email address created by your MySOZO Administrator.

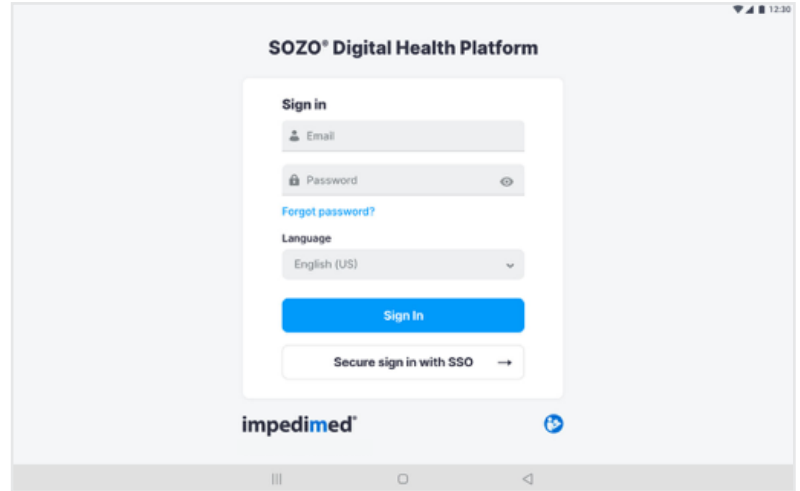

2

## Patient List

1. Provides access to a list of patients in your MySOZO database
2. Allows searching by Last/First name, DOB, sex, MRN and Last assessment date
3. Provides access to individual patient data, by selecting a patient in the patient list

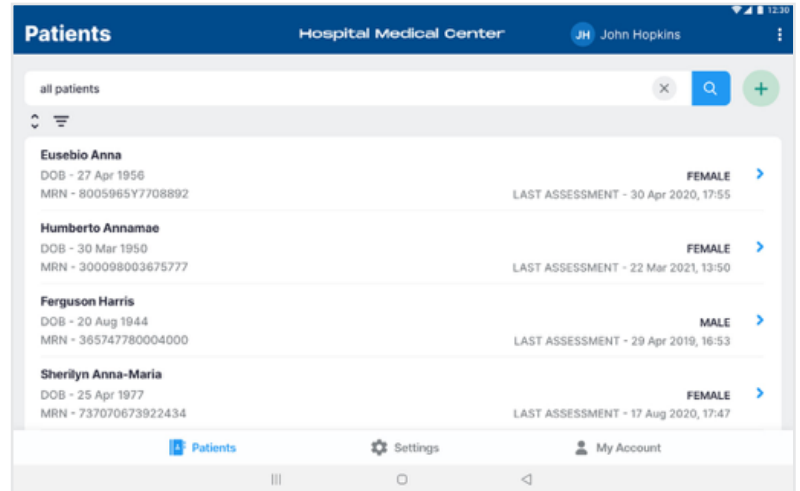


3

## Patient Dashboard

Patient dashboard will give access to:

1. Take a new measurement
2. All measurements taken for the patient along with information about the measurement such as baselines and tags
3. Patient profile
4. Patient notes

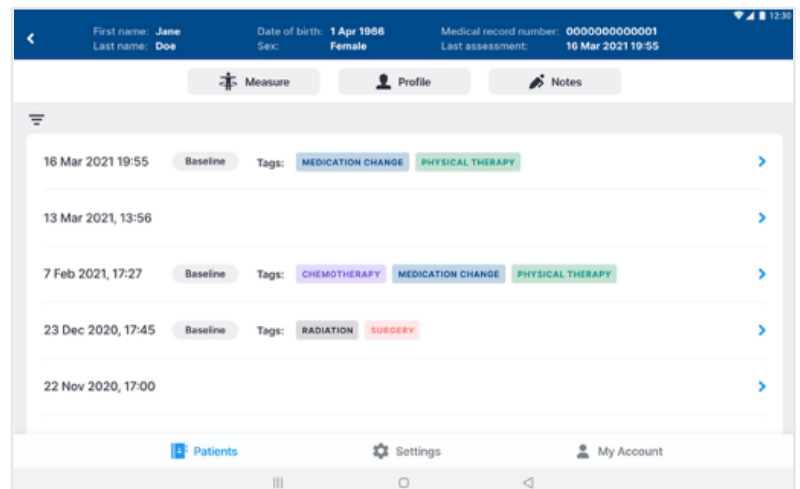

## SOZO APP

# Quick Start Guide

4

## Profile

Patient profile will give access to:

1. Demographics
2. Assessment types available
3. Patient groups

5

## Notes


1. Provides access to a list of notes assigned to the selected patient
2. Ability to create new note
3. Ability to edit or delete an existing note

Please note: A maximum of 10 notes is allowed per patient. To add a new note, first delete an existing note.

6

## Results

1. Provides access to measurement outputs
2. Provides access to historical data
3. Ability to set baseline to customise trending results
4. Ability to assign measurement tags

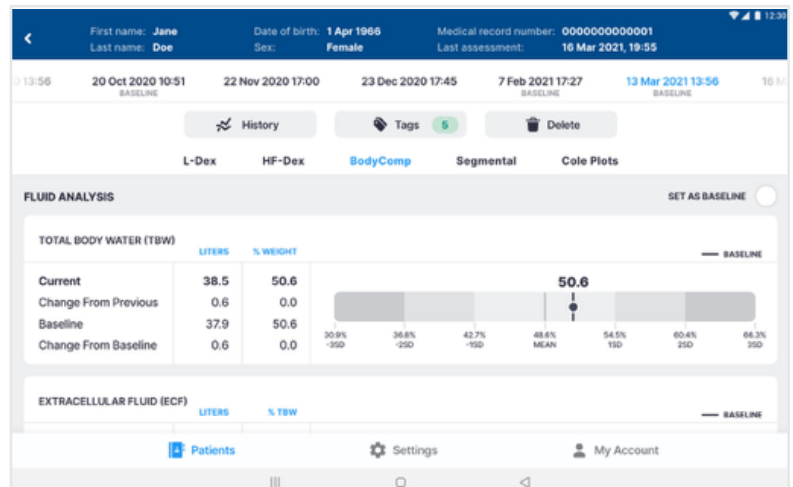


# Quick Start Guide

7

## History

1. Displays graphs of patient measurements overtime.

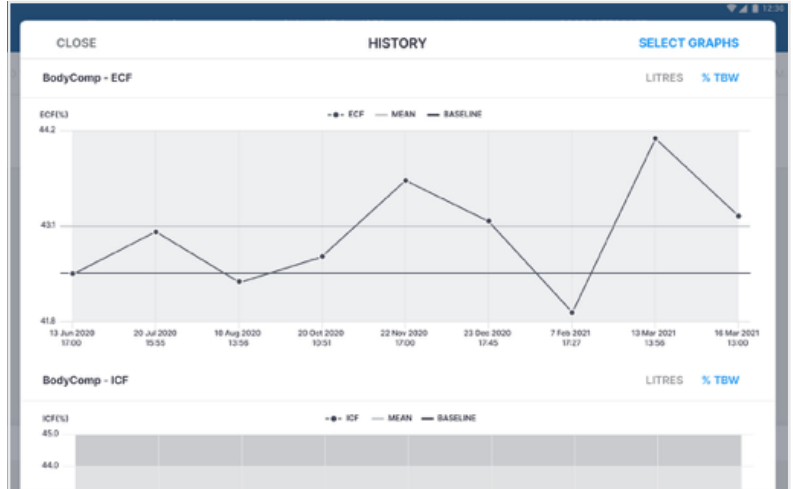

8

## Tags

1. Provides access to a list of tags assigned to the selected measurement
2. Ability to assign new tag
3. Ability to edit or delete an existing tag

Please note: A tag must be first defined by an Administrator before it can be assigned to a measurement.

9

## Restore Patients

1. Provides access to a list of deleted patients
2. Ability to activate a deleted patient