
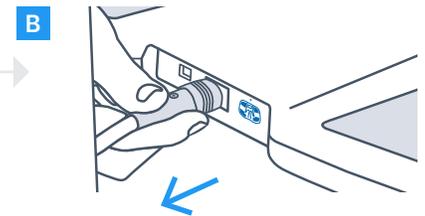
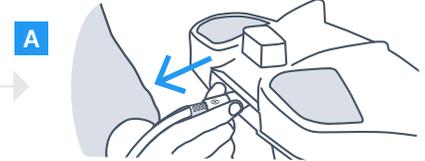


# TAKE MEASUREMENT - SITTING POSITION

1

## Prepare SOZO for sitting position

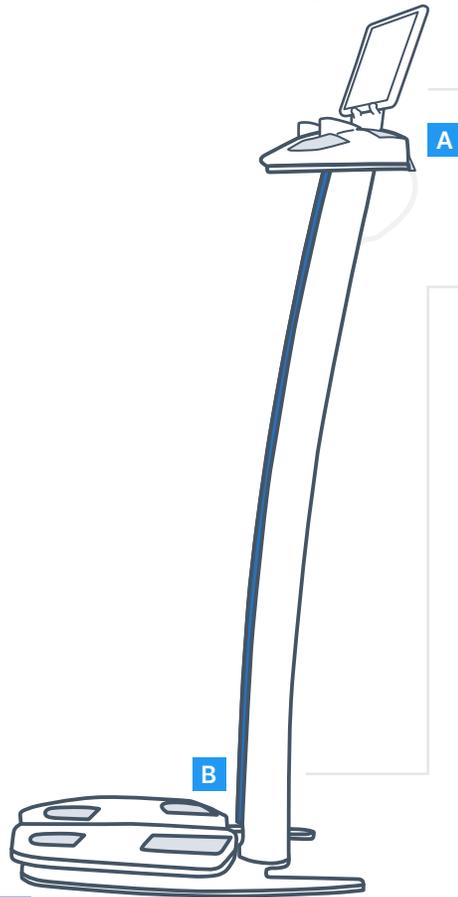

### Remove the cable

Disconnect the power supply, then disconnect the cable from the SOZO hand and foot plates and remove it from the stand.

### Remove SOZOtouch

Remove the SOZOtouch hand plate by sliding it forward and lifting up.

### Remove SOZOstep

Remove the SOZOstep foot plate by sliding it backward and lifting up.

2

## Place SOZO for sitting position

Place the SOZOstep foot plate on the floor. Place the SOZOtouch hand plate on a table across the patient or on a pillow in the patient's lap. Reconnect the cable making sure the screw on the connector is facing up and proceed with the measurement.

**⚠ Do not use a metal table or chair**

### The patient should:

- Sit back in chair with body in balance and shoulders rolled back
- Ensure patient's feet lay flat on the SOZOstep, with equal pressure on both

### The weight limit for SOZO:

**Standing: 375 lb/170 kg**

**Sitting: 750 lb/340 kg**

You may leave the white cord in this position to perform measurements standing and sitting.

**TIP:** Be sure that bare skin is not touching bare skin. **Example:** Patient is wearing shorts and their bare legs are touching each other. **Solution:** Put a towel, paper towel or piece of cloth between the legs.

