
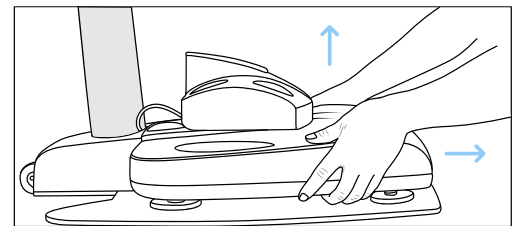
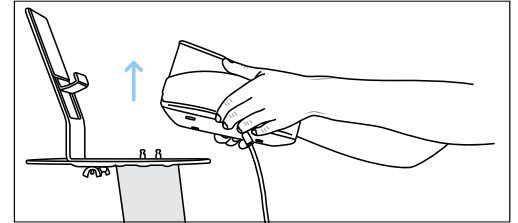


# TAKE MEASUREMENT – SITTING POSITION

## STEP 1

### Prepare SOZO Pro for sitting position

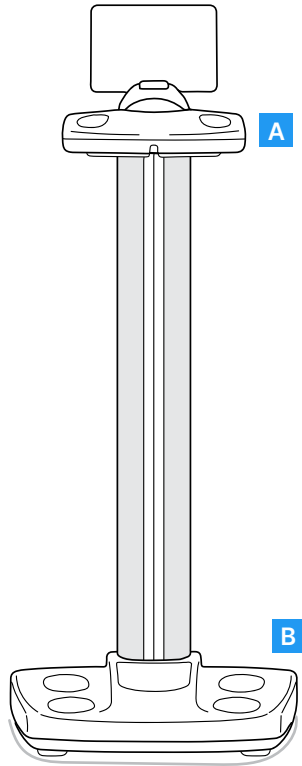

#### Remove the power supply

Disconnect the power supply, then remove SOZO Pro hand unit and foot unit from the stand.

#### Remove hand unit

Remove the hand unit by lifting up.

#### Remove foot unit

Remove the foot unit by sliding it forward.

## STEP 2

### Place SOZO Pro for sitting position

Place the foot unit on the floor. Place the hand unit on a table across the patient or on a pillow in the patient's lap. Reconnect the power supply and proceed with the measurement.

**⚠ Do not use a metal table or chair**

#### The patient should:

- Sit back in chair with body in balance and shoulders rolled back
- Ensure patient's feet lay flat on the foot unit, with equal pressure on both

#### The weight limit for SOZO Pro:

**Standing: 485 lb/220 kg**

**Sitting: 750 lb/340 kg**

The white cord can remain in position A for standing or sitting measurements, or be moved to position B so it exits the back of the hand unit for sitting measurements.

**TIP:** Be sure that bare skin is not touching bare skin. **Example:** Patient is wearing shorts and their bare legs are touching each other. **Solution:** Put a towel, paper towel or piece of cloth between the legs.

